

## Job aid for Unigold HIV1/2 Rapid Test

(For use with whole blood, serum or plasma)

Store kit : 2-30°C

- Check the kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.

This outline is not intended to replace the product insert or your standard operating procedure (SOP).

### 1. Prepare Test

Collect test items and other necessary laboratory supplies.




Remove device from package and label device with client identification number.




### 2. Add Sample

Collect specimen using the disposable pipette.

Add 2 drops (approx. **60µl**) of specimen to the sample port in the device.

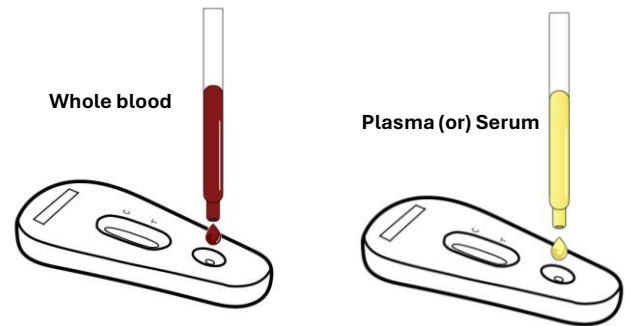

Add 2 drops (approx. **60µl**) of the appropriate wash reagent to sample port vertically.

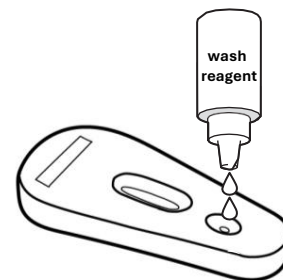


### 3. Read Result

Read test results after **10 minutes** but no later than **12 minutes** incubation time.




**Reactive**

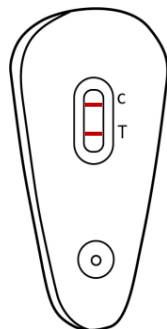


**2 lines** of any intensity appear in **both** the **control** and **test** areas.

**Non-reactive**

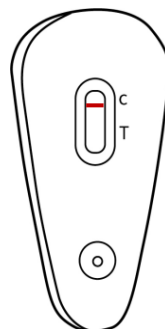

**1 line** appears in the **control** area and no line in test area.

**Invalid**

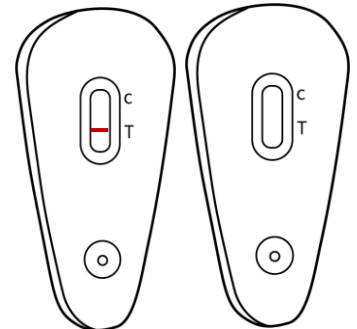

**No lines** appears in the **control** area. Do not report invalid result. Repeat test with new device even if a line appears in the test area.