

## Job aid for Stat-Pak HIV1/2 Rapid Test

(For use with whole blood, serum or plasma)

Store kit : 2-30°C

- Check the kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.


This outline is not intended to replace the product insert or your standard operating procedure (SOP).

### 1. Prepare Test

Collect test items and other necessary laboratory supplies.




Remove device from package and label device with client identification number.

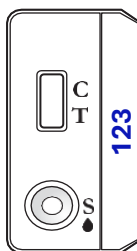


### 2. Add Sample


Collect approximately 5µl of specimen using a new disposable loop or pipette.

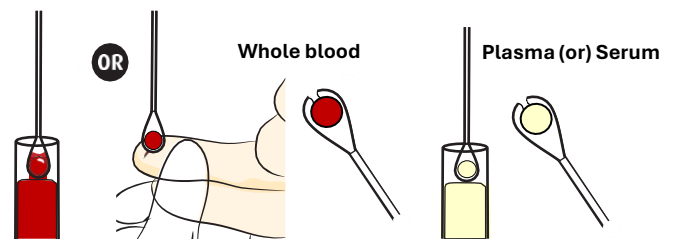

Dispense the sample in the center of SAMPLE well.

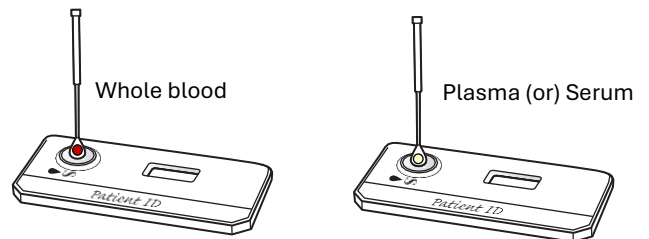

Add 3 drops of buffer, hold vial vertically over the sample well.

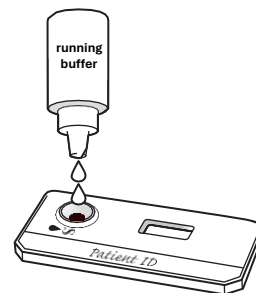


### 3. Read Result

Read the test result after **15 minutes**. Do not read results after **20 minutes**.




**Reactive**

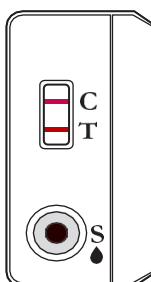


**2 lines** of any intensity appear in **both** the **control** and **test** areas.

**Non-reactive**

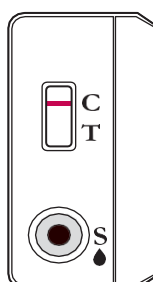

**1 line** appears in the **control** area and no line in test area.

**Invalid**

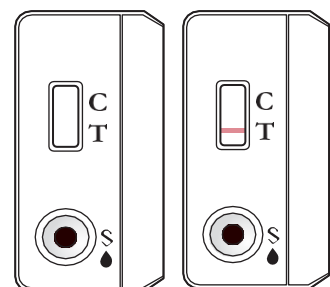

**No lines** appears in the **control** area. Do not report invalid result. Repeat test with new device even if a line appears in the test area.