

Job aid for Stat-Pak HIV1/2 Rapid Test

(For use with whole blood, serum or plasma)

Store kit : 8-30°C

- Check the kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.


This outline is not intended to replace the product insert or your standard operating procedure (SOP).

1. Prepare Test

Collect test items and other necessary laboratory supplies.




Remove device from package and label device with client identification number.

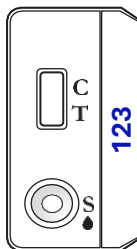


2. Add Sample


Collect approximately 5µl of specimen using a new disposable loop or pipette.

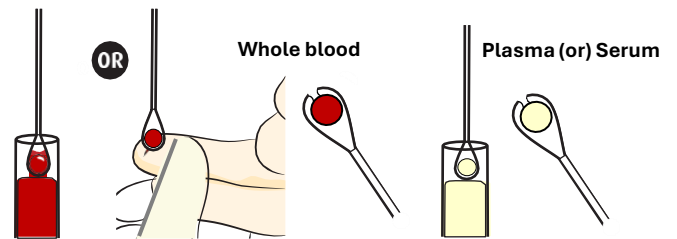

Dispense the sample in the center of SAMPLE well.

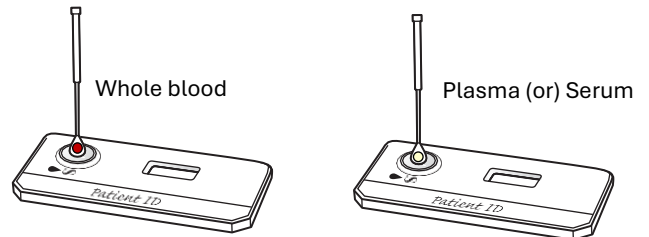

Add 3 drops of buffer, hold vial vertically over the sample well.

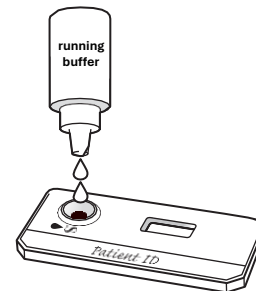


3. Read Result

Read the test result after **15 minutes**. Do not read results after **20 minutes**.




Reactive

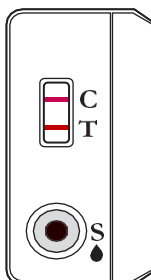


2 lines of any intensity appear in **both** the **control** and **test** areas.

Non-reactive

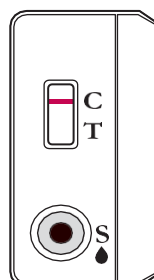

1 line appears in the **control** area and no line in test area.

Invalid

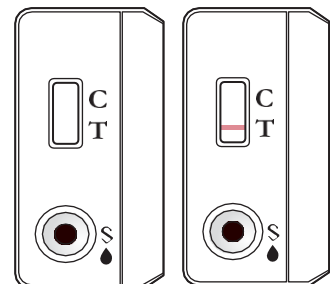

No lines appears in the **control** area. Do not report invalid result. Repeat test with new device even if a line appears in the test area.